**DENTSPLY**  
CAULK

**Aquasib Ultra**  
**XLV**

SMART WETTING® IMPRESSION MATERIAL

**CE**

**Directions For Use**

English .....	3
Español .....	8
Français .....	14
Deutsch .....	20
Italiano .....	25
Português .....	31
Dansk .....	36
Nederlands .....	41
Svenska .....	46

## Directions for use of Cartridge Dispensed Aquasil Ultra XLV Regular Set Smart Wetting® Impression Material

### Classification

Quadrafunctional Hydrophilic Addition Reaction Silicone Impression Material  
Type 3: Light-Bodied Consistency

### COMPOSITION AND DESCRIPTION

Aquasil Ultra XLV Smart Wetting® **Regular Set** Impression Material is a quadrafunctional hydrophilic addition reaction silicone, light body, elastomeric impression material with excellent hydrophilic properties, dimensional accuracy, high tear strength, and resistance to permanent deformation. Aquasil Ultra XLV **Regular Set** Impression Material with Smart Wetting® improves both wetting on the tooth surface and model detail reproduction. Aquasil Ultra XLV **Regular Set** Impression Material is available in cartridge delivery. It complies with the requirements of ISO 4823 for dental elastomeric impression materials.

**Caution:** For dental use only.

### INDICATION FOR USE

Aquasil Ultra XLV **Regular Set** Impression Material is suitable for:

- All impression techniques where a light body material would be desired by the operator.
- Dual phase full arch impression technique with Aquasil Ultra Monophase **Regular Set**, Aquasil Ultra Heavy **Regular Set** or Aquasil Soft Putty Impression Materials.
- Dual phase quadrant arch impression technique with Aquasil Ultra Heavy **Regular Set**, Aquasil Ultra Monophase **Regular Set** or Aquasil Soft Putty Impression Materials.
- Any combination can also be used for the duplication of models.

Classification ISO 4823	<i>Aquasil Ultra XLV: Type 3: Light-bodied consistency</i> <i>Aquasil Ultra LV: Type 3: Light-bodied consistency</i> <i>Aquasil Ultra Monophase: Type 2: Medium-bodied consistency</i> <i>Aquasil Ultra Heavy: Type 2: Medium-bodied consistency</i>
Working Time *	2:15- 2:45 min
% Recovery from deformation	> 98.0%
% Strain in compression	<i>Aquasil Ultra XLV: 4-5</i> <i>Aquasil Ultra LV: 3-4</i> <i>Aquasil Ultra Monophase: 3-4</i> <i>Aquasil Ultra Heavy: 2-3</i>
Detail Reproduction	< 20 microns
Linear dimension change	< 0.50%
Setting Time (Mouth removal time)	5:00 min from start of mix
Mixing ratio by volume	1 part base to 1 part catalyst
* Ta instruments AR-1000 Oscillating Rheometer	

Data on file

## CONTRAINDICATIONS

Do not use Aquasil Ultra XLV **Regular Set** Impression Material as a temporary reliner.  
Do not use with a polyether, polysulfide or condensation silicone material.

## WARNINGS

1. Avoid prolonged or repeated exposure of the Aquasil Ultra impression materials with skin and eyes. Irritation and possible corneal damage may result. Skin rash or other allergic reactions may result in susceptible individuals.

**Eye and Skin Contact:** Flush eyes with flowing water for 15 seconds and consult a physician. Flush skin with flowing water for 15 seconds, then wash area with soap and water after contact. Flush oral tissues with copious amounts of water. Consult a physician if rash persists.

## PRECAUTIONS

1. This product is intended to be used only as specifically outlined in the Directions For Use. Any use of this product inconsistent with the Directions For Use is at the discretion and sole responsibility of the practitioner.
2. Aquasil Ultra XLV **Regular Set** Impression Material should not be intermixed with, or used in conjunction with other manufacturer's vinyl polysiloxanes.
3. Aquasil Ultra **Regular Set** Impression Materials should not be intermixed with Aquasil Ultra **Fast Set** Impression Materials.
4. Do not contaminate retraction cords with latex of "sulfur-containing" polymeric gloves. Handle cords with college pliers. Do not touch prepared tooth with gloves.
5. If the preparation is treated with hydrogen peroxide, rinse with copious amounts of water before making the impression.
6. Variations in the relative amounts of base and catalyst will neither significantly increase nor decrease work or set times. Optimal performance requires a ratio of 1:1.
7. **Storage and Shelf Life:** Store and use Aquasil Ultra impression materials between 10°-24°C (50°-75°F) and in relative humidity of 50% ± 20%. Do not use after expiration date. Work time may be affected by temperature. See step by step instructions.

## ADVERSE REACTIONS

1. Corneal damage may result with prolonged eye exposure to the tray adhesive or the impression material. (See Warnings statement)
2. Allergic contact dermatitis and other allergic reactions may occur in susceptible individuals. (See Warnings and Precautions statements)
3. Eye redness, tearing and blurred vision may result from direct exposure. Vomiting, nausea, diarrhea, and irritation of the mouth, throat and stomach may result if ingested. Skin irritation may result from direct exposure and skin defatting from prolonged exposure. (See Warnings and Precautions statements)

## STEP BY STEP INSTRUCTIONS

### Cartridge Dispensing Gun Instructions

Hold the release lever up and pull the plunger all the way back in the dispenser handle.

#### Cartridge Loading

1. Open cartridge lock by lifting up top clasp.
2. Orient and insert cartridge with v-shaped notch facing down.
3. Close the top clasp to lock cartridge into dispenser gun.
4. Remove the cartridge cap by turning 90 degrees counter clockwise. The cartridge cap can be replaced on cartridge when storing the cartridge after initial use or the used mixing tip can be left in place until the next use to serve as a self-sealing cap.
5. Dispense a small amount of base and catalyst before installing mix tip to ensure even flow from cartridge. Use gentle pressure. Be sure no plug is present. If any plug blocks flow, clear with an instrument. Wipe away excess from the cartridge.
6. Install a mixing tip on the cartridge by lining up the v-shaped notch on the outside colored rim of the mix tip with the v-shape notch on the cartridge flange. If mix tip does not seat easily, be sure the internal end view of the mix tip is aligned correctly. The mix tip itself can turn in its cap. The two holes should line up with the v-shaped piece that is on the outside rim of the mix tip cap to facilitate inserting. Turn tip to align in its cap.
7. When mix tip colored rim v-notch is lined up with the v-shaped notch on the cartridge flange, turn colored mix tip cap 90 degrees in a clockwise direction to lock in place on cartridge.

#### Dispenser Operation

1. Squeeze dispenser trigger with moderate and even pressure to begin mixing material. Material will stop flowing after trigger is released.
2. Automixed Aquasil Ultra XLV **Regular Set** Impression Material may be dispensed directly to the mouth with provided mix tips. The intraoral tip can be inserted into the end of the mix tip (stator) for direct intraoral syringing.

#### OR

An intraoral syringe may be filled by extruding the mixed material directly into the syringe by back filling.

**For subsequent usage**, remove the used mixing tip and squeeze the dispenser trigger to extrude (bleed) material from the two openings of cartridge. Wipe end of cartridge. Install a new mixing tip and twist one quarter turn to lock to position.

**To remove the cartridge**, hold the release lever up and withdraw plunger in a backward direction. Dispose of empty cartridge.

#### Technique with a Rigid Tray or with Double Arch Technique

1. Select and/or prepare a suitable metal tray, a firm disposable tray or a custom tray. Use rigid trays of sufficient size to provide at least 2-3 mm thickness of impression material.
2. Brush a thin layer of Tray Adhesive (Silfix Tray Adhesive) onto tray and allow to air dry for five minutes. Adhesive should be used even with perforated trays.
3. If there is any debris or blood, clean the field with air water spray.
4. Remove excess water with suction. **Do not desiccate the tooth.**

5. Backfill the impression syringe with Aquasil Ultra XLV **Regular Set** Impression Material or attach an intraoral tip to the end of the mix tip for direct intraoral syringing.
- 6a. Dispense selected Aquasil Ultra **Regular Set** tray material or Aquasil Soft Putty directly into impression tray. Do not layer the material or place it into rows. This technique ensures the least amount of air incorporation.
- 6b. **It is suggested to simultaneously syringe completely around the margins of the preparation, making sure to keep intraoral tip within impression material.** Inject syringe material into any existing anatomy and continue syringing around the preparation until it is completely covered with syringe material. Do not blast with air. Aquasil Ultra Impression Material's hydrophilic properties improve adaptation to the tooth enabling better "wetting".
7. Seat loaded tray. Retain impression in position until firmly set. (**NOTE:** The material sets faster in the mouth than on the bench). Check impression **in the mouth** (*not on the bench*) to be sure it is firm, resilient, and non-tacky before removal. The minimum removal time is 5 minutes from start of mix. At 72°F (22°C) Aquasil Ultra **Regular Set** Impression Materials have a minimum work time of 2 minutes 15 seconds and a minimum removal time of 5 minutes from start of mix. Variations in temperature from that stated above will affect work and set times. **Higher temperatures reduce work times and lower temperatures increase them.**
8. Remove impression by pulling slowly to break seal, then snap out along the long axis of the tooth. Rinse impression under cold water and blow dry. Do not store impression in direct sunlight. If the impression is to be shipped, use suitable packaging to preclude distortion.

## **DISPENSER CLEANING INSTRUCTIONS**

The dispenser may be cleaned by scrubbing with hot water and soap or detergent. Dispensers or used tips, if left in place for storage, exposed to spatter or spray of body fluids or that may have been touched by contaminated hands, or oral tissues, should be disinfected with a hospital-level disinfectant. Sodium hypochlorite (5.25%), isopropyl alcohol and dual or synergized quaternaries are approved disinfectants. Water-based disinfectant solutions are preferred. Some phenolic based agents and iodophor based products may cause surface staining. Agents containing organic solvents, such as alcohol may tend to dissolve the plastic dispenser and plunger. The disinfectant manufacturer's directions should be followed properly for optimum results.

**Technique Tip:** Unset Aquasil Ultra Impression Material can be removed from clothing with dry cleaning solvent.

## **DISINFECTION**

The impression should be disinfected with a hospital-level disinfectant. Iodophors, sodium hypochlorite (5.25%), chlorine dioxide, and dual or synergized quaternaries are approved disinfectants. To disinfect polyvinyl siloxane material, thoroughly soak by spraying or immerse the impression in any recommended hospital level disinfectant except neutral glutaraldehyde for the contact time recommended by the disinfectant manufacturer for optimum results. Water-based disinfectant solutions are preferred.

## **CASTING AND ELECTROPLATING**

The impression should be removed from the disinfectant, rinsed with water and exposed to air to dry for at least the same amount of time it was exposed to the disinfectant solution before it is poured. The impression may be poured up to fourteen days. The material is compatible with a range of die stones. Impressions may be silver or copper-plated.

## **LOT NUMBER AND EXPIRATION DATE**

1. Do not use after expiration date.
2. The following numbers should be quoted in all correspondences.
  - Reorder number
  - Lot number on the cartridge
  - Expiration date

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© 2002 DENTSPLY International Inc. All rights reserved.
Manufactured in the U.S.A. by
DENTSPLY DeTrey GmbH