DENTSPLY
DETREY

Seal&Protect®

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Česky

Seal&Protect

Protective Sealant for Exposed Dentine

Seal&Protect is a protective sealant for exposed root dentine.

Seal&Protect reduces wear of cervical dentine by infiltrating and coating the dentine thus increasing its mechanical strength.

Seal&Protect reduces cervical hypersensitivity by blocking dentinal tubules.

Seal&Protect reduces dentine decalcification by acids and has an antimicrobial effect against caries-associated bacteria.

For dental use only.

PRESENTATION FORM

Seal&Protect is available in a specially designed solvent-proof laminated bottle. Regular contents are 4.5 ml. The trial size bottle contains 1 ml only.

Alternatively, Seal&Protect is available pre-dosed for single patient use, each quix™ - unit dose containing 0.15 ml.

COMPOSITION

- Di- and trimethacrylate resins
- PENTA (dipentaerythritol penta acrylate monophosphate)
- Functionalised amorphous silica
- Photoinitiators
- Butylated hydroxytoluene
- Cetylamine hydrofluoride
- Triclosan
- Acetone

INDICATIONS

- Reduction of abrasion, erosion and abfraction of exposed cervical dentine
- Reduction of plaque accumulation
- Treatment of hypersensitive cervical areas

CONTRAINDICATIONS

Seal&Protect should not be used for patients with a known hypersensitivity to any of the components of the sealant.

SIDE EFFECTS

In very rare cases, reversible inflammatory changes of the oral mucosa have been reported after accidental contact with acetone solutions and acrylate monomers. Therefore, avoid contact of Seal&Protect with mucous membranes.

WARNINGS AND PRECAUTIONS

1. Seal&Protect contains acetone. Acetone is highly flammable. Keep away from sources of ignition - no smoking. Do not breathe vapour. Take precautionary measures against static discharges.
2. Seal&Protect is harmful to the eyes when in direct contact. In case of contact, rinse immediately with plenty of water and seek medical advice.
3. Seal&Protect contains methacrylates and triclosan which may be irritating to oral tissues and skin and which may cause sensitisation in susceptible persons. Minimise contact with oral mucosa. After contact, rinse with water or water/air spray by using suction. After contact with skin, wash immediately with plenty of soap and water.

ADVERSE REACTIONS

The following adverse reaction has been associated with the use of acetone solutions and acrylate monomers:

- Reversible inflammatory changes of the oral mucosa after accidental contact.

APPLICATION

1. Cleanliness is paramount for the development of a protective layer of Seal&Protect. Clean the dentine surface with a rubber cup and pumice or a prophylaxis paste (Nupro®).
2. Remove prophylaxis paste with air/water spray. Isolate area to be treated with cotton rolls. In the case of mandibular teeth, place saliva ejector. Dry cleaned area with a 2-second blow of air free of oil or water contamination. Avoid desiccating the dentine, leave a moist, but not wet glistening surface.
3. Dispense Seal&Protect into a fresh DENTSPLY Applicator Dish¹ or standard dappen dish. For handling of quix - unit dose, please see illustrated instructions.
Per surface to be treated, two to three drops are required. It is recommended to treat no more than three adjacent surfaces per application procedure.
4. Immediately after dispensing apply ample amounts of Seal&Protect to the dentine surface with a DENTSPLY Applicator Tip¹. Thoroughly saturate the exposed dentine.
5. Leave the dentine surface undisturbed for 20 seconds.
6. Remove excess solvent by blowing gently with air for a few seconds from a dental syringe.
7. Cure Seal&Protect for 10 seconds using a curing light.
8. Apply a second layer of Seal&Protect.
9. Remove excess solvent from the second layer by blowing gently with air from a dental syringe.
10. Cure Seal&Protect for 10 seconds using a curing light.
11. Remove oxygen-inhibited (soft surface) layer with a cotton pellet or cotton roll. Check for excess material in the gingival sulcus and remove if any, with a periodontal probe.
12. Discharge used DENTSPLY Applicator Tip and DENTSPLY Applicator Dish.

PATIENT RECALLS

It is recommended that all treated surfaces be checked for partial or total loss of the sealant during a second appointment. Depending on individual oral hygiene, wear of the sealant will require reapplication every three to six months.

STORAGE

Do not store above 24 °C. Keep out of direct sunlight.

Keep Seal&Protect in a well ventilated place.

The Seal&Protect bottle should be tightly closed immediately after use.

BATCH NUMBER AND EXPIRY DATE

The batch number should be quoted in all correspondence which requires identification of the product.

Do not use after expiry date.

If you have any questions, please contact:

Manufacturer:
DENTSPLY DeTrey GmbH
De-Trey-Str. 1
78467 Konstanz
GERMANY
Phone +49 (0) 75 31 5 83-0

Distributor:
DENTSPLY Limited
Hamm Moor Lane
Addlestone, Surrey
Weybridge KT15 2SE
Phone (0 19 32) 85 34 22

© DENTSPLY DeTrey 2000-12-11

¹ DENTSPLY Applicator Dish and Applicator Tips are available from your dental dealer.

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